
**SCOOP STRETCHER X RAY
TRANSPARENT 10XLP**



User Manual



PROMEBA
EMERGENCY & RESCUE

Applicable scope

It is mainly used for hospitals, sports, ambulances and battlefields that carry patients and injured people.

Features

1. Made of PE material, separable structure, with clutch devices at both ends, the stretcher can be separated into 2 right and left parts, without the need to move the patient in its original place, the patient can be quickly inserted or removed from below. The body reduces secondary injuries to the patient.
2. Concave surface, so that patients can be on the firm stretcher.
3. The length of the stretcher can be adjusted to three different positions depending on the patient's height.
4. It is characterized by being lightweight, safe and easy to use for sterilization.

Technical parameters (Tolerance $\pm 5\%$)


Size (L×W×H)	Folding size (L×W×H)	Maximum load	Usable Length Adjustment
214×42×7cm	167×42×7cm	180KG	167~200cm

Method of use

1. According to the patient's height, take out the black button on both sides, keep it upright (Picture 4), stretching or shortening the length (Picture 2), the length of the stretcher can be adjusted to three different positions, each position corresponds to a hole (Picture 3), choose the appropriate position, release the button in the level state (Picture 2), keep the lock button completely locked.

Hole 0 is the initial position, hole 1 is position 1, hole 2 is position 2, hole 3 is position 3 (note: positions 1 and 2 are for normal use). The length of the initial position H0 = 167 cm, 1 position H1 = 178 cm, 2 position H2 = 189 cm, 3 position H3 = 200 cm. (Picture 1)

WARNING: When adjusting the length of the stretcher to prohibit more than 3 positions, that is, the length is 200 cm.

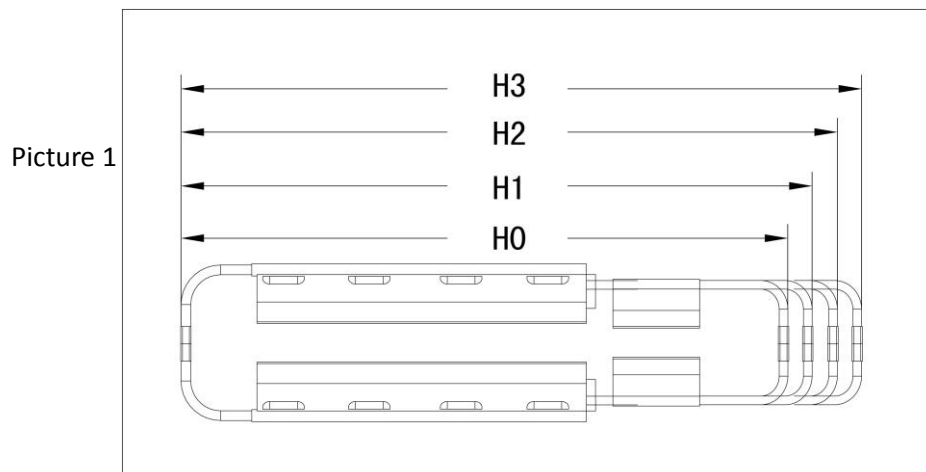




Picture 2




Picture 3

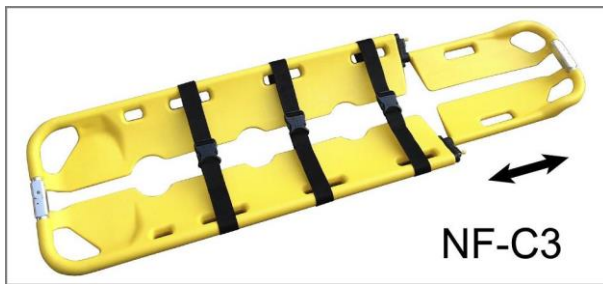


Picture 4

2 、 Press the clutch button on both ends of the stretcher, so that the stretcher can be separated into two parts. (Picture 5, Picture 6)

Warning: make sure the clutch button is in the locked state when transporting patients.

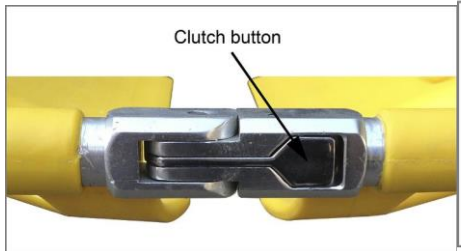


Picture 5

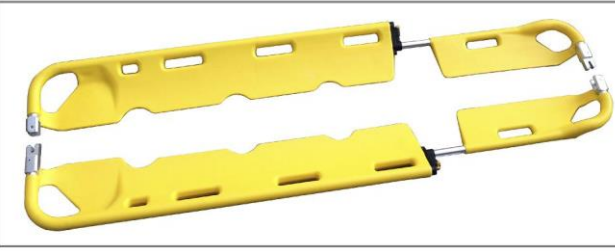

Picture 6

3. After use, it can be stored as folded. Stretch the length to the maximum state and then fold (Picture 7).

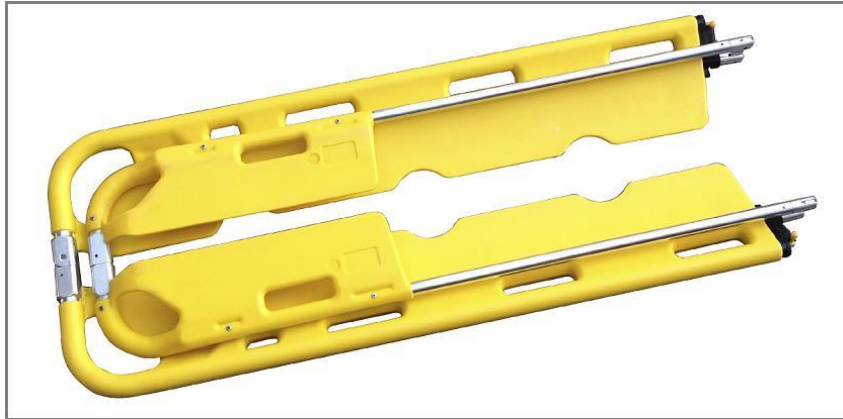

Picture 7

Precautions

1. When transporting, make sure that the clutch buttons on both ends are locked, wear safety belts.
2. After adjusting the length, make sure the position is locked, do not use a stretcher without blocking to avoid harming patients again. When the length of the stretcher is adjusted to prohibit more than 3 positions, that is, the length is 200 cm.

Maintenance

1. Keep clean as a routine (incl. Disinfect)
2. Check if there are any loose parts or not as usual.

Storage and transportation

1. Store this stretcher in place of humid and non-corrosive environments.
2. Be transferred by common transport tools.

Packaging and accessories

1. Packed in high durability cardboard.
2. Accessories: 3pcs seat belts, user manual.