



MASK Catalogue



Summary

- . Start right. Stay right.
- . Mask portfolio at a glance.
- . ResMed Premium Mask Selector
- . For Her

FULL FACE MASKS

- . AirFit™ F10
- . Quattro™ Air
- . Quattro™ FX
- . Mirage Quattro™
- . Ultra Mirage™
- . Mirage™ Liberty

NASAL PILLOWS MASKS

- . AirFit™ P10
- . Swift™ FX Bella
- . Swift™ FX
- . Swift™ LT

NASAL MASKS

- . AirFit™ N10
- . Swift™ FX Nano
- . Mirage™ FX
- . Mirage™ Activa LT
- . Mirage™ SoftGel
- . Mirage™ Micro

PEDIATRIC MASKS

- . Pixi™

NON-VENTED MASKS

- . Quattro™ Air NV
- . Quattro™ FX NV
- . Ultra Mirage™ Full Face mask NV
- . Ultra Mirage™ Nasal Mask NV

HOSPITAL MASKS

- . Full face mask Vented
- . Full face mask Non-Vented
- . Nasal mask

MASK ACCESSORIES

- . Bella™ Loops
- . Gecko™ Nasal Pad
- . CPAP Wipes

PRACTICAL INFO

- . Care and cleaning
- . Technical information
- . Technical innovation

Start right. Stay right.

ResMed offers more than just mask solutions for you and your patients. We offer a lasting partnership, based on our long legacy of proven innovation and our ongoing commitment to service and support.

ResMed is committed to getting it right from the start.

Our aim is improved quality of life through a good night's sleep. Making sure everything is right early on in the patient-mask relationship is essential to staying on therapy. Education, mask set-up, fitting, as well as compliance monitoring all require a significant time and cost investment.


- Research shows that mask satisfaction is a key factor to staying on PAP therapy. In fact, 63% of respondents who reported that they stopped PAP therapy said they did so because they found their mask was uncomfortable or they did not like wearing¹ it.
- ResMed's simple, comfortable, easy-to-use masks with proven innovative technology are designed to fit the first time, giving patients the best possible start.

Once we start to make a difference in your patients' sleep life, we'll never stop.

Your time is incredibly valuable, and frequent and unnecessary mask replacements burden you with extra work and additional product and service costs. As your patients' sleep mask partner, we strive for lifelong patient satisfaction - we want every night to be ResMed right.

- A ResMed mask means an easy, comfortable fit, not just the first time, but every time.
- ResMed offers high-quality and high performance masks that bring superior long-lasting value and all are backed by a responsive and proactive service and support team.


¹DVL Smith 2013: The study, conducted August-September 2012, surveyed sleep apnoea patients in the USA, UK, Germany, France and Australia. The study was sponsored by ResMed.

<p>FULL FACE MASKS</p>	 <p>AirFit™ F10</p>	 <p>Quattro™ Air</p>	 <p>Quattro™ FX</p>	 <p>Mirage Quattro™</p>	 <p>Ultra Mirage™</p>	 <p>Mirage™ Liberty</p>
<p>NASAL PILLOWS MASKS</p>	 <p>AirFit™ P10</p>	 <p>Swift™ FX</p>	 <p>Swift™ LT</p>	 <p>Swift™ FX Bella</p>		
<p>NASAL MASKS</p>	 <p>AirFit™ N10</p>	 <p>Swift™ FX Nano</p>	 <p>Mirage™ FX</p>	 <p>Mirage™ Activa LT</p>	 <p>Mirage™ SoftGel</p>	 <p>Mirage™ Micro</p>
<p>PEDIATRIC MASKS</p>	 <p>Pixi™</p>					
<p>NON VENTED MASKS</p>	 <p>Quattro™ Air NV</p>	 <p>Quattro™ FX NV</p>	 <p>Ultra Mirage™ FFM NV</p>	 <p>Ultra Mirage™ Nasal NV</p>		
<p>HOSPITAL MASKS</p>	 <p>Hospital FFM</p>	 <p>Hospital FFM NV</p>	 <p>Hospital Nasal</p>			

ResMed Premium Mask Selector

Does the patient breathe through their mouth while sleeping?

If YES

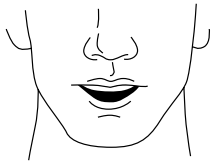

select a
full face mask




AirFit™ F10

or if **classic fit**
preferred



Quattro™ Air

If NO

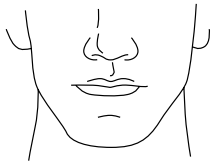

select a
nasal pillows



AirFit™ P10

or if **nasal mask**
preferred



AirFit™ N10

ResMed's for Her series of masks, designed just for women.

for **HER**

ResMed has created a number of masks specifically to meet the needs of women. By recognising the difference, we can bring a new level of fit, comfort, choice and personal expression to women with sleep apnea.

We want to offer you the best of both worlds - innovation and technology combined with the kind of personalisation that recognises your individuality.



AirFit™ P10

Nasal pillows mask for Her



AirFit™ N10

Nasal mask for Her



AirFit™ F10

Full face mask for Her



Swift™ FX Bella

Nasal pillows system





AirFit™ F10

Full face mask

Full face benefits in a compact size

ResMed's AirFit™ F10 full face mask is part of ResMed's AirFit range – the right fit for you. Its lightweight, streamlined design enhances comfort and ease-of-use, while its headgear and frame work in synergy, to deliver the same performance and stability you have grown to trust in a ResMed traditional full face mask.

Superior comfort

AirFit F10's sleek, lightweight design gives patients a clear line of sight so they can read, watch TV or even wear glasses while wearing the mask. In fact, 75% of patients found AirFit F10 to be more comfortable than their current ResMed traditional full face mask.¹

Proven performance

The dual-wall Spring Air™ cushion provides a secure seal for better performance. When asked about AirFit F10's seal and stability, the majority of patients we spoke to, told us that it outperformed their ResMed traditional full face mask.¹

Ease-of-use

The mask's four-part design makes it easy for you to demonstrate and for patients to take apart, clean and reassemble. Its intuitive design seamlessly integrates into your patient's bedtime routine.

¹ResMed internal study of existing ResMed patients, conducted between 25 July and 4 September, 2013 comparing their ResMed traditional full face mask with AirFit F10.




AirFit™ F10

Full face mask

Component

Part Number

Headgear (blue)	(S)	63165
	(Std)	63164
	(L)	63166
Headgear for Her	(S)	63167
	(XS)	63736
	(S)	62737
Cushion	(M)	62738
	(L)	62739
	(XS/S)	63137
	(M/L)	63138
Elbow with swivel		62762

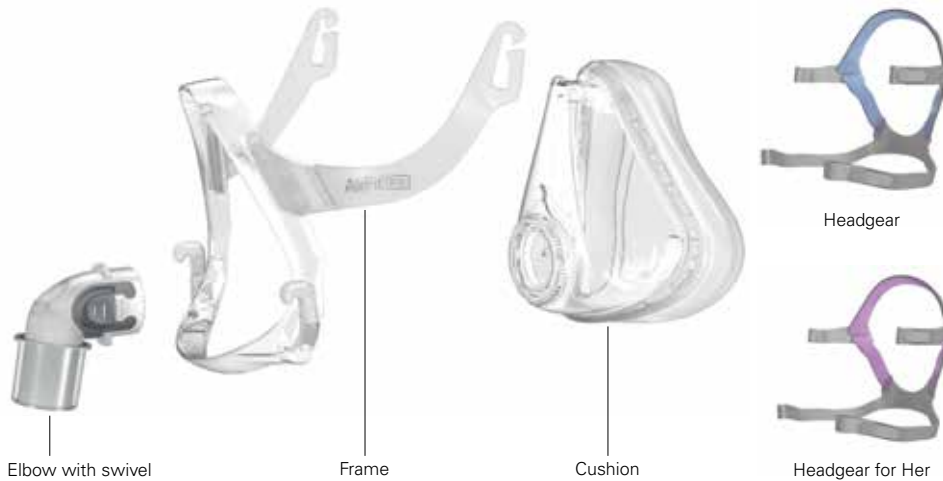

	AirFit™ F10				AirFit for Her		
Size	X-Small	Small	Medium	Large	X-Small	Small	Medium
EUR 1	63104	63105	60106	63107	63142	63143	63144
EUR 2	-	63109	63110	63111	63145	63146	63147
EUR 3	-	63113	63114	63115	63148	63149	63150



Quattro™ Air
Full face mask

The lighter way to better sleep

ResMed's new lightweight Quattro Air Full Face Mask builds on Quattro's proven performance and reliability. It provides enhanced comfort and ease of use, and is 45% lighter¹ than the previous Mirage Quattro mask.

Builds on the strength of Quattro

Quattro Air provides performance you can rely on, in a new sleeker, lighter mask with less bulk on the face, so it's bound to have strong patient appeal.

Less parts for quick and easy set-ups

Quattro Air has fewer parts than most other Full Face Masks (FFM) - just four. It's quick to demonstrate and easy to train patients to use, supporting patient compliance.

Improved patient usability

The mask features greatly reduced complexity with improved comfort and stability, reducing patient call-backs and freeing up more of your time.

Innovative design for easy acceptance

It's light, streamlined and comfortable, allowing you to assist those of your patients who struggle to stick with therapy.

¹ The Medium size of Quattro Air is 45% lighter than the Medium size of Mirage Quattro.





Quattro Air Headgear



Quattro Air for Her Headgear



Elbow with Swivel



Frame



Cushion

Quattro™ Air

Full face mask

Part		ResMed Part No
Quattro™ Air Headgear	(Std)	62756
	(S)	62757
Quattro™ Air for Her	(Std)	62759
	(S)	62758
Elbow with Swivel		62762
Frame	(XS/S)	62761
	(M/L)	62761
Cushion	(XS)	62736
	(S)	62737
	(M)	62738
	(L)	62739

Size	Quattro™ Air				Quattro™ Air for Her		
	X-Small	Small	Medium	Large	X-Small	Small	Medium
EUR 1	62704 ¹	62705	62706	62707	62743	62744	62745
EUR 2	-	62709	62710	62711	62746	62747	62748
EUR 3	-	62711	62714	62715	62749	62750	62751

¹Not available in all countries.



ResMed

Quattro™ FX

Full face mask

Therapy has never been easier

Bringing light, unobtrusive design to a full face mask, Quattro FX provides an effective alternative to conventional full face masks and more choice in full face therapy.

Contoured cushion membrane around sensitive nasal bridge area

Rolls and inflates to gently meet the nose rather than stretch over it.

Clear line of sight and feeling of openness

A smaller frame also makes side-sleeping much more comfortable.

Built-in vent design

Disperses air gently and quietly

Flexible, durable Spring frame strong enough to stabilise the mask on the face

Soft enough to accommodate user movement during sleep.

360° elbow rotation

Positions the tubing for ease of use.

Swivel

Offers a convenient way to connect and disconnect air tubing.

Dual-wall, Spring Air™ cushion

Provides soft, even pressure distribution.

Set-and-forget headgear clips easy to grip for attaching and detaching

Retain desired headgear settings.




Quattro FX



Quattro FX
for Her



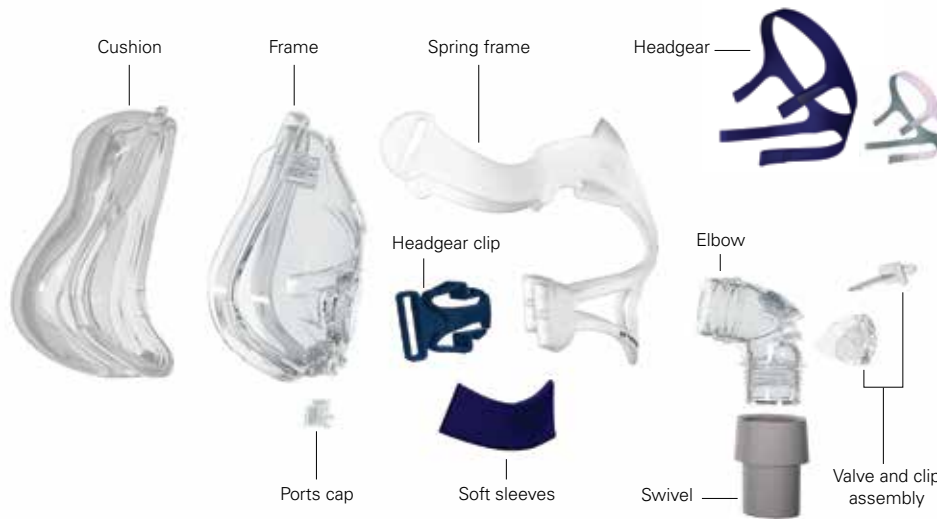

Size ¹	Quattro™ FX			Quattro™ FX for Her	
	Small	Medium	Large	Small	Medium
EUR 1	61703	61704	61705	62518	62519
EUR 2	61706	61707	61708	62520	62521
EUR 3	61709	61710	61711	62522	62523

¹ All Quattro FX mask systems come with the (M) headgear.

Quattro™ FX

Full face mask

Component	Part Number
Swivel	16565
Ports cap	(x2) 16570
	(x10) 16571
Elbow	61282
Valve & clip	(x1) 61286
	(x10) 61287
Elbow assembly	61294
Headgear clips (blue), lower	61353
Cushion	(S) 61721
	(M) 61722
	(L) 61723
Frame	(S) 61730
	(M) 61731
	(L) 61732
Headgear	(S) 61734
	(M) 61733
	(L) 61738
Soft sleeves (blue)	61758
Spring frame	(S) 61759
(soft sleeves included)	(M) 61760
	(L) 61761
Quattro FX for Her	
Headgear	(S) 62506
	(M) 62507
Spring frame	(Std) 62510
(soft sleeves included)	(M) 62511
Soft sleeves (pink)	62516
Headgear clips (grey)	62515



ResMed

Mirage Quattro™
Full face mask

Fourth generation technology

Mirage™ dual-wall cushion with Spring Air™ technology in combination with the revolutionary MicroFit™ dial achieves a good mask fit - first time, every time.

MicroFit™ dial

Offers 24 positions for maximum fit, seal and comfort.

Mirage™ dual-wall cushion with Spring Air™ technology

Optimises seal thanks to built-in elasticity of the cushion.

Moving cushion membrane at chin and nasal bridge

Maintains seal and accommodates mouth opening and jaw drop.

360° one snap elbow with quick release swivel

For easy removal from the mask and flexible tube positioning.

Built-in anti-asphyxia valve (AAV)

For safety if airflow to the mask is impeded.

Cushion clip

With tabs, allows easy removal of cushion from frame.

Built-in vents

Minimise noise and maximise CO₂ washout.


Easy-access pressure and oxygen ports

Provide convenient monitoring and delivery of oxygen.

Set-and-forget headgear clips

For ease of refit.



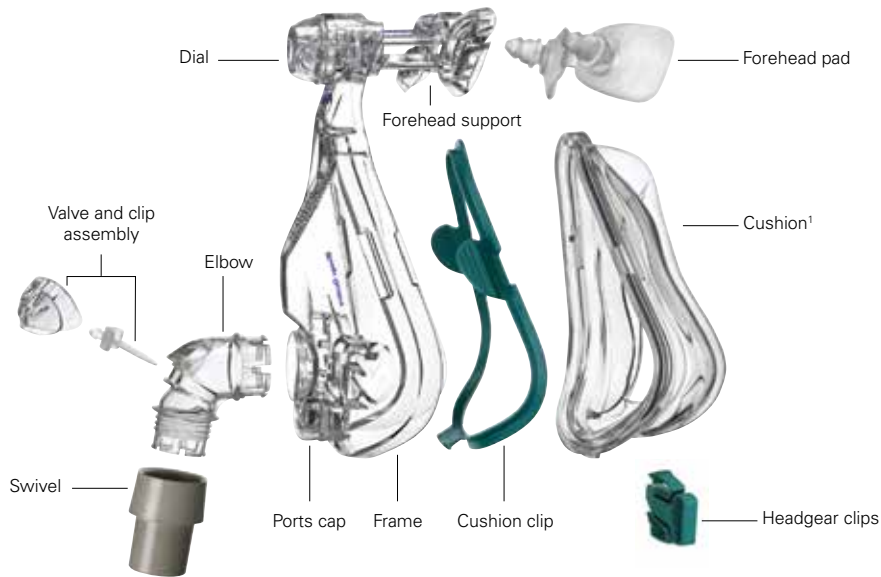

Mirage Quattro™				
Size ³	X-Small	Small	Medium	Large
EUR 1	61204	61205	61206	61207
EUR 2	61208	61209	61210	61211
EUR 3	61228	61229	61230	61231

¹ Mask cushions are supplied with mask cushion clip.

² Small (16118) and Large (16119) headgear come without clips.

³ Standard headgear supplied with Medium and Large. Small headgear supplied with X-small and Small.

Mirage Quattro™

Full face mask

Component	Part Number
Swivel	16565
Forehead pad	(x1) 60123
	(x10) 60124
Frame	(XS) 61210
	(S) 61271
	(M) 61272
	(L) 61273
Cushion clip	(XS) 61274
	(S) 61275
	(M) 61276
	(L) 61277
Elbow	61282
Valve & clip assembly	(x1) 61286
	(x10) 61287
Forehead support	61288
Dial	61286
Cushion and clip	(XS) 61290
	(S) 61291
	(M) 61292
	(L) 61293
Ports cap	(x2) 61295
	(x10) 61296
Elbow assembly	61294
Headgear ²	(S) 16118
	(Std) 16733
	(L) 16119
Headgear clips	(x2) 60115
	(x10) 60116
Emergency release cord assembly	60618



Ultra Mirage™
Full face mask

The solution for mouth leak

The Ultra Mirage Full Face Mask covers both the nose and mouth so patients receive effective therapy even in the presence of mouth breathing, a common problem for CPAP and bilevel users.

Mirage dual-wall cushion

For proven comfort and seal.

Built-in anti-asphyxia valve (AAV)

For safety if airflow to the mask is impeded.

Adjustable forehead support and pads

Provide maximum stability and comfort.

Air vent

Provides excellent CO₂ washout to minimise re-breathing.

Easy-access pressure and oxygen ports

Provide convenient monitoring and delivery of oxygen.

360° quick-release swivel

Allows easy disconnection from tubing.


Four quick release headgear clips

Allow headgear to be removed without refitting.



Ultra Mirage™

Full face mask

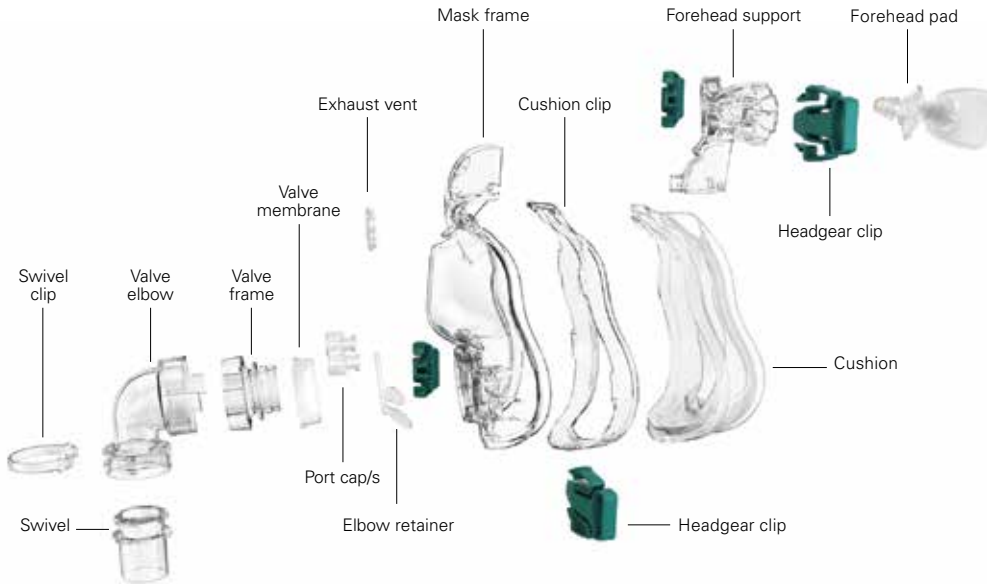

Ultra Mirage™ 1, 2						
Size	Small Standard	Small Shallow	Medium Standard	Medium Shallow	Large Standard	Large Shallow
EUR (except UK& FR)	60610	60611	60612	60613	60614	60615

¹ Includes elbow retainer, valve frame, membrane, elbow, swivel clip and swivel.

² Includes 5 elbow clips and 5 swivel clips.

³ Standard headgear supplied with all sizes of complete mask assembly.

Component		Part Number
Cushion	(S-Std)	16604
	(M Std)	16605
	(L-Std)	16606
	(S-Sh)	16671
	(M Sh)	16672
	(LSh)	16673
Emergency		60618
Valve membrane	(x1)	16617
	(x10)	16618
Cushion clip	(S)	16674
	(M)	16675
	(L)	16676
Swivel		16689
Headgear	(S)	16118
	(Std) ³	60674
	(L)	16119
Headgear clips	(x2)	60115
	(x10)	60116
Forehead pad		30123
	(x10)	60124
	(S)	60670
Frame	(M)	60671
	(L)	60672
		60673
Swivel clip		60680
Elbow retainer		16637
Port cap/s	(x2)	60681
	(x10)	60682
Elbow retainer		60683
Air vent	(x1)	16509
	(x10)	16510
Anti-asphyxia valve		16613
(includes elbow assembly)		



Mirage™ Liberty

Full face mask

Full face freedom

Mirage Liberty is the alternative to conventional full face masks, offering a sense of freedom, while providing comfort and stability in a lightweight mask.

Dual-wall nasal pillows

Provide an independent seal between nose and mouth.

Mirage dual-wall cushion

For proven comfort and seal.

360° one snap elbow

For easy removal from the mask and flexible tube positioning.

Built-in anti-asphyxia valve (AAV)

For safety if airflow to the mask is impeded.

Built-in vents

Minimise noise and maximise CO₂ washout.

Easy-access pressure and oxygen ports

Provide convenient monitoring and delivery of oxygen.

Set-and-forget headgear clips

For ease of refit.

Breathe-O-Prene® headgear with stabilising elements

Contours to different facial shapes and optimises stability.



Nasal pillows¹



Mouth cushion




Mask frame



Headgear clip

Ports cap

Elbow assembly
(elbow, valve and clip)

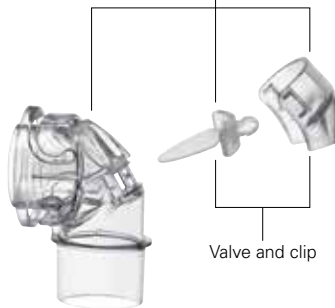

Valve and clip



Headgear

Clips
(upper L&R)

	Mirage Liberty™	
Size ¹	Small ²	Large ²
EUR 1	61302	61303
EUR 2	61304	61305
EUR 3	61314	61315

¹ Complete mask assembly includes all sizes of nasal pillows (S, M & L).

² Standard headgear size supplied with all sizes of complete mask assembly.

Mirage™ Liberty

Full face mask

Component		Part Number
Ports cap	(x2)	16570
	(x10)	16571
Mouth cushion	(S)	61330
	(L)	31331
Nasal pillows	(S)	61333
	(M)	61334
	(L)	61335
Mask frame	(S)	61337
	(L)	61338
Elbow assembly		61340
Valve and clip	(x1)	61342
	(x10)	61343
Headgear assembly	(S)	61348
	(Std) ²	61349
Swivel assembly		60014
Inlet tube wih swivel		60926
	(x10)	60927
Clips, upper (L&R)		61352
Clips, lower (L&R)		61353



NASAL PILLOWS
MASKS



A quieter mask to fit your patients

ResMed's AirFit™ P10 nasal pillows mask is part of ResMed's AirFit series - the right fit for your patients. This is our quietest mask yet, offering the reliability and performance of the Swift™ FX in a design that's 50% quieter and around 50% lighter.

Lighter

It's almost 50% lighter than our most popular pillows mask¹ and designed to provide a comfortable night's sleep.

Easy to Fit

QuickFit™ elastic headgear automatically adjusts to each patient's head. It can be fitted in under 30 seconds².

Quieter

QuietAir™ woven-mesh vent is whisper quiet at just 21.4 dBA³, for minimal patient and partner disturbance.

3-Part Simplicity


Key-clip nasal pillows feature colour coded sizing and left/right orientation for fast and easy fitting.

¹ ResMed Clinical Study FECS3. ResMed data (pressure, leak, average daily usage, AHI) collected from a sample size of 21 participants.

² ResMed Clinical Study FECS3. 90% of participants fitted the mask comfortably in less than 30 seconds.

³ Declared dual numbered noise emission values (with uncertainty 3dBA) in accordance with ISO 4871.



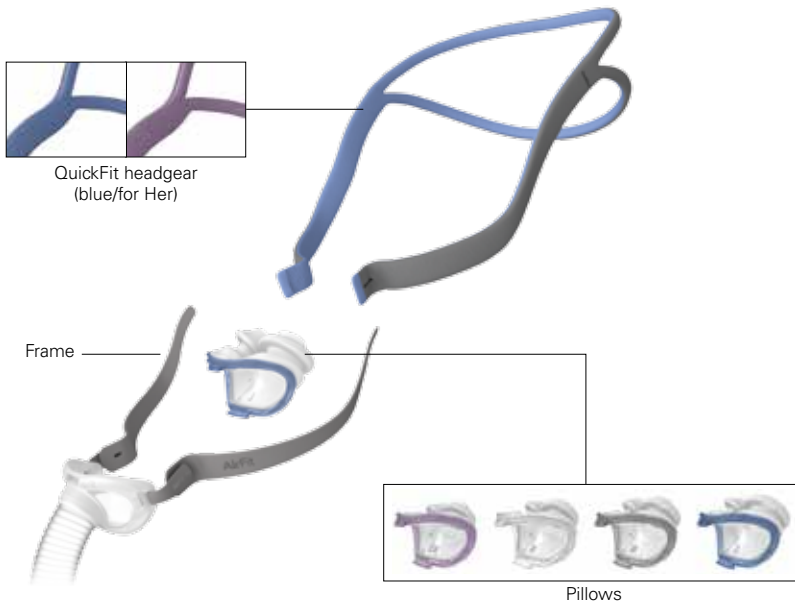

AirFit™ P10

Nasal pillows mask

Component		Part Number
QuickFit headgear (blue)	(Std)	62935
QuickFit headgear (pink)	(S)	62936
Frame		62938
Pillows	(XS)	62930
	(S)	62931
	(M)	62932
	(L)	62933

	AirFit P10 ¹	AirFit P10 for Her ²
EUR 1	62901	62911
EUR 2	62902	62912
EUR 3	62903	62913

¹S, M and L pillows are included with the mask and standard size headgear (blue).

²XS, S and M pillows are included with the mask and small size headgear (pink).



ResMed

Swift™ FX Bella

Nasal pillows system

Style and design with her in mind

Now there's an extra choice for everyone in the FX range with Swift™ FX Bella & Bella Gray - an unobtrusive design that's easy to fit, light, soft and streamlined.

Intuitive, quick to fit

Once the right pillow size is selected, Swift FX Bella Series requires minimal or often no adjustment. So it saves you time, money and additional patient visits.

Interchangeable with Swift FX

Swift FX Bella Series is fully interchangeable with Swift FX (both use the same pillows and tubing) for an increased level of patient compliance.

Easier therapy acceptance

Designed for instant appeal, Swift FX Bella Series is not daunting, helping users accept and continue therapy. And with very few parts, the system is easy to clean and assemble.


Stable seal

The Swift FX Bella & Bella Gray system stays in place and maintains seal effectively.

Hair management

The exclusive Bella headgear is designed to stay out of patients hair for uncomplicated hair management and the entire system enhances comfort.



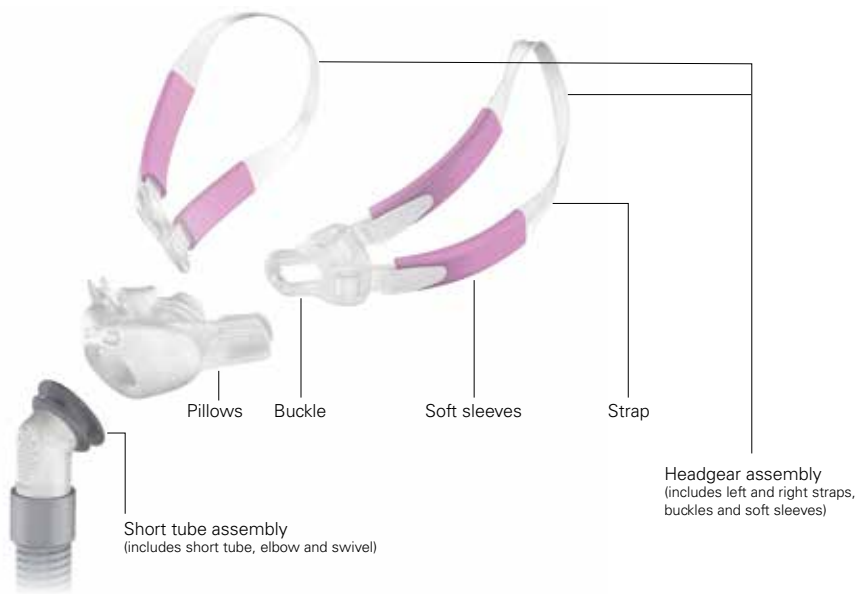

Swift™ FX Bella

Nasal pillows system

Component		Part Number
Frame system (no headgear)	(XS)	61510
	(S)	61511
	(L)	61512
Pillows	(XS)	61520
	(S)	61521
	(M)	61522
	(L)	61523
Short tube assembly	(x1)	61528
	(x10)	61527
Headgear assembly (pink)		61581
Headgear assembly (gray)		61582
Headgear and Swift FX headgear Combo (pink)		61580
Headgear and Swift FX headgear Combo (gray)		61583

	Swift™ FX Bella	Swift™ FX Bella Gray
EUR 1	61561 ¹	61569 ²

¹Includes XS, S, and M pillows, Swift FX Bella headgear assembly, Swift FX for Her headgear assembly, pink backstrap and pink soft wraps pillows available separately.

²Includes S, M and L pillows, Swift FX Bella Gray headgear assembly, Swift FX headgear assembly, blue backstrap and gray soft wraps.

There's Nothing Hard About It

The Swift FX is so soft, light and simple, it feels like it's barely there! Minimal contact on the face and unobtrusive design make therapy less intimidating and easy to accept.

Intuitive, almost instant fit

Once the right pillows size is selected, it requires minimal or often, no adjustment.

Stable seal at high pressures

The system stays in place and maintains seal effectively even at higher treatment pressures¹

Easier therapy acceptance by patients

Designed for instant appeal, Swift FX is non- intimidating to facilitate therapy acceptance.

Reduced inventory, wide (>95%) fit range²

Packed with three pillow sizes to suit most users.
An Extra-Small size is also available.

Easy product handling

With very few parts, Swift FX is easy to clean and assemble.

Convenient to recommend or prescribe

User guide and online support enable easy training for users and staff.

¹ Tested up to 20 cm H₂O in clinical trials.

² Based on ResMed's market research.





	Swift™ FX¹	Swift™ FX for her²
EUR 1	61501	61545
EUR 2	61502	61546
EUR 3	61503	61567

¹Includes S, M and L pillows and standard size headgear (blue).

²Includes XS, S and M pillows and small size headgear (pink).

Swift™ FX

Nasal pillows system

Component		Part Number
Pillows	(XS)	61520
	(S)	61521
	(M)	61522
	(L)	61523
Short tube assembly	(x1)	61528
	(x10)	61527
Headgear assembly (blue)		61529
Soft wraps (grey)		61530
Headgear assembly (pink)		61543
Soft wraps (pink)		61544



ResMed

Swift™ LT
Nasal pillows system

Less is more

A nasal pillows technology suitable for side sleeping, offering minimal contact with the face and optimising mask stability. It offers a flexible fit and personalised comfort.

Nasal pillows

Provide a gentle and comfortable seal.

Rotating pillow sleeve assembly

Enables adjustments of the cushion angle.

Interchangeable tube with rotating elbow and swivel

Offers a range of sleeping positions for flexibility and comfort.

Diffuse vent design

Disperses air gently and quietly away from the bed partner.


Packed with 3 pillow sizes - S, M, L

For reduced inventory.

Comfortable and stable Breathe-O-Prene® headgear

Contours to different facial shapes.



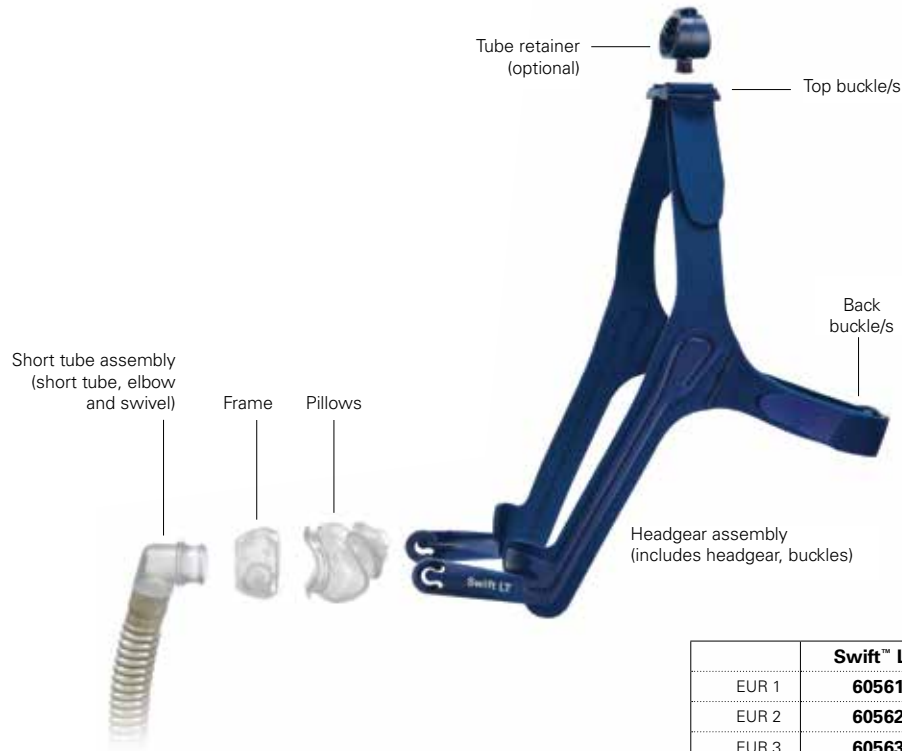

Swift™ LT

Nasal pillows system

Component		Part Number
Back buckle/s	(x1)	60558
	(x10)	60559
Pillows	(XS)	60574
	(S)	60571
	(M)	60572
	(L)	60573
Frame		60575
Short tube assembly		60577
Headgear assembly		60578
Top buckle/s	(x1)	60579
	(x10)	60580
Tube retainer		60586

	Swift™ LT ¹
EUR 1	60561
EUR 2	60562
EUR 3	60563

¹Product packed with S, M and L pillows.
XS pillows sold separately.



**NASAL
MASKS**



AirFit™ N10
Nasal mask

Effortless performance

ResMed's AirFit N10 compact nasal mask is part of ResMed's new AirFit range - the right fit for you. Its excellent performance, combined with optimal usability, raises the standard in the compact nasal category. The AirFit N10 is simple and intuitive for patients to fit and comfortable to wear, with an under-eye frame providing clear line of sight.

Visual freedom

With a compact design and a sleek frame that's positioned underneath the eyes, the AirFit N10 provides patients with unrestricted vision.

Comfort

The soft cushion, SoftEdge™ headgear and optional soft sleeves all contribute to patient satisfaction. And circular diffused venting gently disperses exhaled air.

Simplicity

Self-aligning EasyClick™ clips secure the frame to the headgear easily. Plus it has a simple clip-in cushion and a durable yet highly flexible integrated short tube.



AirFit N10



AirFit N10
for Her





AirFit™ N10

Nasal mask

Component		Part Number
Headgear (blue)	(Std)	63260
	(S)	63262
Headgear for Her	(S)	63261
	(Std)	63240
Cushion	(S)	63241
	(W)	63242
	(Std)	63270
Frame	(Std)	63270
Headgear clip	(x10)	63280

Size	AirFit™ N10			AirFit™ N10 for Her
	Small	Medium	Wide	Small
EUR 1	63227¹	63209	63211	63210
EUR 2	-	63212	63214	63213
EUR 3	-	63215	63217	63216

¹Not available in all countries.



Swift™ FX Nano

Nasal mask

Easy breathing compact mask

It comes with a soft, low-profile nasal cushion and minimal headgear that really opens up the face. It's time for your patients to slip into something more comfortable.

Builds on the strength of Swift™ FX

Swift™ FX Nano delivers the compact size, facial freedom, reliability and comfort that the Swift name is famous for with the breathing ease of a nasal cushion.

Set up once and they're set

It fits quickly and requires minimal or often no adjustment once fitted, saving time, money and additional patient visits.

Simple to use and reassemble after cleaning

Swift™ FX Nano has fewer main parts than most other nasal masks (just three: headgear, cushion and tube) and it is simple to use.


Maintains a comfortable seal

Its smooth 360° rotating ball-joint helps maintain seal effectively and provides comfort with adaptive flexibility.

Designed for easy user acceptance

Swift™ FX Nano is light and compact with minimal headgear helping encourage patients to accept and continue therapy.



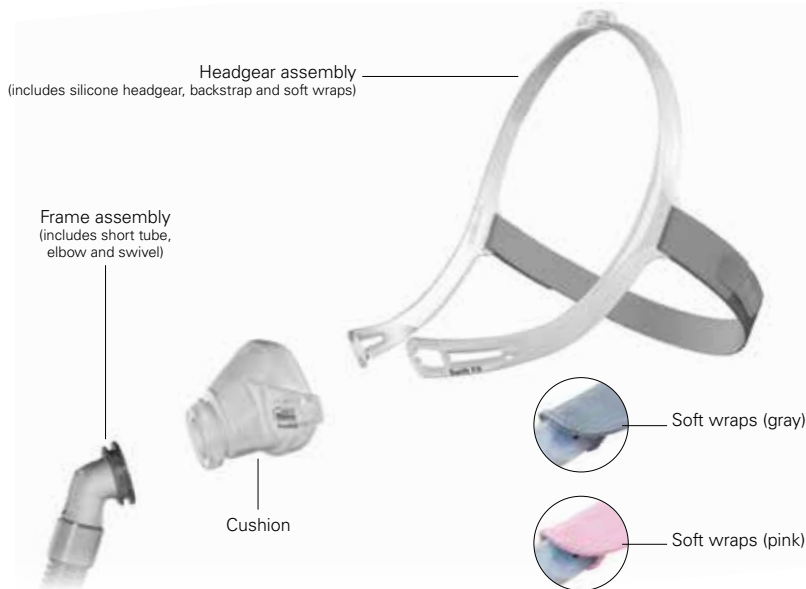

Swift™ FX Nano

Nasal mask

Component		Part Number
Headgear assembly (includes silicone headgear, backstrap and soft wraps)	Gray (Std)	62248
	Pink (S)	62248
Cushions	(Std)	62230
	(W)	62281
	(S)	62231
Frame assembly (gray)	(x1)	62236
	(x10)	62237
Soft wraps (gray)	(x2)	61530
Soft wraps (pink)	(x2)	61544

	Swift™ FX Nano		Swift™ FX Nano for Her
Size	Standard ¹	Wide ¹	Small ²
EUR 1	62202	62253	62203
EUR 2	62204	62255	62257
EUR 3	62203	62205	62207

¹Includes Cushion (Standard or Wide),
Frame Assembly (gray), and Swift FX Nano
Headgear Assembly (gray).

²Includes Cushion (Small), Frame Assembly (pink),
and Swift FX Nano for Her Headgear Assembly (pink).



Mirage™ FX
Nasal mask

One fit. One solution. One for all.

The perfect balance of sophisticated design and easy functionality, Mirage™ FX is the solution you've been waiting for: performance, ease-of-use and quality all rolled into one.

Flexible, winged forehead support

Allows a dynamic range of fit and adjustment.

Lightweight, durable frame

Strong enough to stabilise the mask on the face, soft enough to accommodate user movement during sleep.

Mirage™ dual-wall cushion with Spring Air™ technology

Provides even pressure distribution, with extra softness at the nasal bridge, reducing common discomfort in this sensitive area.

Diffuse vent design

Disperses air gently and quietly, away from bed partner.

Headgear loops

Simplify attachment and detachment of headgear; retain optimal headgear settings, ready to fit and wear.

Squeeze-tab elbow

Makes it easy to attach and detach the elbow from the frame.

SoftEdge™ headgear design

Headgear design is kind to the face, reducing marks and discomfort.





Mirage™ FX

Nasal mask

Component		Part Number
Headgear (gray)	(Std)	62110
Headgear (gray)	(S)	62138
Cushion	(Std)	62111
Frame	(Std)	62113
Elbow assembly		62114
Headgear for Her (pink)	(S)	62129
Cushion for Her	(S)	62136
Frame for Her	(S)	62143

	Mirage™ FX ¹	Mirage™ FX for Her ²
EUR 1	62100	62132
EUR 2	62101	62133
EUR 3	62102	62139

¹Includes standard size cushion, standard size frame, and standard size headgear (gray).

²Includes small size cushion, small size frame, and small size headgear (pink).



Mirage™ Activa LT

Nasal mask

Made to fit different facial shapes

A nasal mask from ResMed with unique ActiveCell™ Technology offers performance and comfort to suit user's movement.

Mirage™ dual-wall cushion with ActiveCell™ Technology

Inflates to suit user's movement and varying pressures.

MicroFit™ dial

Offers 24 positions for maximum fit, seal and comfort.

Cushion clip

With tabs, allows easy removal of cushion from frame.

360° one snap elbow with quick release swivel

For easy removal from the mask and flexible tube positioning.

Diffuse vent design

Disperses air softly and quietly away from bed partner.

Set-and-forget headgear clips

For ease of refit.

Easy-access pressure and oxygen ports

Provide convenient monitoring and delivery of oxygen.

Comfortable and stable Breathe-O-Prene® headgear

Contours to different facial shapes.



Mirage™ Activa LT

Nasal mask

Component		Part Number
Headgear	(S) ²	16118
	(Std)	16733
	(L) ²	16119
Elbow		16387
Forehead support		16393
Elbow assembly		16399
Swivel		16565
Headgear clips	(x2)	16569
	(x10)	16734
Ports cap	(x2)	16570
	(x10)	16571
		16571
Forehead support pad	(x1)	60123
Cushion clip	(x10)	60124
Cushion	(Std)	60176
	(S)	60197
Cushion (includes Cushion clip)	(S)	60198
	(M)	60177
	(L)	60178
	(LW)	60179
Mask frame	(S)	60193
	(Std)	60180
Dial		61289



	Mirage™ Activa LT			
Size	Small ^{1, 3}	Medium ^{1, 4}	Large ^{1, 4}	Large Wide ^{1, 4}
EUR 1	60186	60151	60152	60153
EUR 2	60187	60154	60155	60156
EUR 3	60188	60157	60158	60159

¹Standard headgear supplied with all sizes of complete mask assembly.

²Small (16118) and large (16119) headgear come without clips.

³Small frame supplied with small complete mask assembly.

⁴Standard frame supplied with medium, large and large-wide mask system.



Instant softness, lasting comfort

The DoubleGel™ cushion, with its clear gel layer provides comfort, while the blue gel layer maintains support and stability. Backed by ResMed's renowned performance and quality, the frame accommodates two interchangeable cushions - Mirage™ SoftGel and Mirage™ Activa LT.

Unparalleled softness and comfort

The DoubleGel™ cushion's outer layer provides unparalleled softness. The blue gel layer maintains stability and support. Fitting is a simple task-for both current users and new patient.

Ultra-quiet therapy

Special array of small, built-in vents diffuses air quietly. That means a good night's sleep for patients and their bed partners.

Correct fitting made easier

Combination of the Mirage dual-wall cushion with the DoubleGel™ cushion design makes fitting simple. The cushion adapts to individual facial contours, creating a comfortable fit with ease.

ConvertAble™ mask means freedom of choice

ResMed ConvertAble masks offer the choice of two different cushions on one frame. Freedom of choice helps patients accept their therapy early and stay engaged in treatment.



Mirage™ SoftGel

Nasal mask



Component		Part Number
Headgear	(S) ²	16118
	(Std)	16733
	(L) ²	16119
Elbow		16387
Forehead support		16393
Elbow assembly		16399
Swivel		16565
Headgear clip	(x2)	16569
	(x10)	16734
Ports cap	(x2)	16570
	(x10)	16571
Forehead support pad	(x1)	60123
Forehead support pad	(x10)	60124
Mask frame	(S)	60193
	(Std)	60180
Cushion clip	(Std)	60176
	(S)	60197
	(S)	61631
Cushion and cushion clip	(S)	61631
	(M)	61632
	(L)	61633
	(LW)	61634

Size	Mirage™ SoftGel ¹			
	Small	Medium	Large	Wide
EUR 1	61605	61606	61607	61608
EUR 2	61636	61637	61638	61639
EUR 3	61610	61611	61613	61614

¹Standard headgear supplied with all sizes of complete mask assembly.

²Small (16118) and large (16119) headgear come without clip.



Mirage™ Micro

Nasal mask

Proven seal, perfect fit

A nasal mask from ResMed that provides personalised fit, seal and comfort for more users.

Mirage dual-wall cushion

For proven comfort and seal.

MicroFit dial

Offers 24 positions for maximum fit, seal and comfort.

Clip-free cushion

Allows easy removal of cushion from frame.

Diffuse vent design

Disperses air softly and quietly away from bed partner.

360° one snap elbow with quick release swivel

For easy removal from the mask and flexible tube positioning.

Elbow partition

Aids CO₂ washout and reduces noise output.


Easy-access pressure and oxygen ports

Provide convenient monitoring and delivery of oxygen.

Comfortable and stable Breathe-O-Prene® headgear

Contours to different facial shapes.



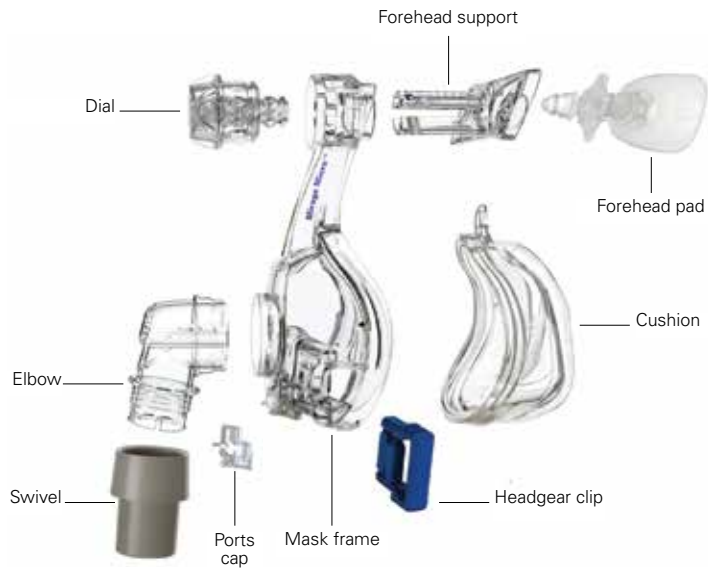

Mirage™ Micro

Nasal mask

Component		Part Number
Headgear	(S)	16118
	(Std) ²	16733
	(L)	16119
Elbow		16387
Forehead support		16393
Elbow assembly ³		16399
Swivel ¹		16565
Headgear clips	(x2)	16569
	(x10)	16734
Ports cap	(x2)	16570
	(x10)	16571
	(x1)	60123
Forehead support pad	(x10)	60124
Cushion	(S) ¹	16388
	(M)	16389
	(L)	16390
	(LW)	16391
	(XL)	16392
Mask frame	(Std)	16394
	(S) ¹	16395

Mirage™ Micro					
Size	Small	Medium	Large	Large Wide	X-Large
EUR 1	16338	16339	16340	16341	16342
EUR 2	16343	16344	16345	16346	16347
EUR 3	16348	16349	16350	16351	16352

¹Small frame supplied with small complete mask assembly.

²Standard headgear supplied with small complete mask assembly.

³Elbow assembly includes elbow and swivel.



PEDIATRIC
MASKS



ResMed

Pixi™
Paediatric mask

Designed with children and their families in mind

Pixi is not merely a scaled down adult mask. Starting with a 'clean slate' a child's bone structure, facial characteristics and skin sensitivity were all used to develop this unique paediatric mask.

Comfortable Breath-o-prene® headgear

Unobtrusive design with three points of adjustment for easier fitting.

Soft, comfortable cushion

Single-wall, thin silicone cushion minimises pressure on children's soft faces, without compromising seal.

Two tubing positions

Air tubing can be moved to either side of the cushion depending on your child's sleeping position.

Oxygen/sleep study port

Allows connection of oxygen or pressure tubing for monitoring direct to the mask.

Spring-flex lightweight tube

Stretches and bends easily, so it won't get caught or drag as your child moves around during sleep.


Quick-release latch

Makes it easy to take the mask off, while retaining desired headgear settings.

Emergency latch

Release the latch even faster in an emergency by pulling firmly on the lower headgear strap.



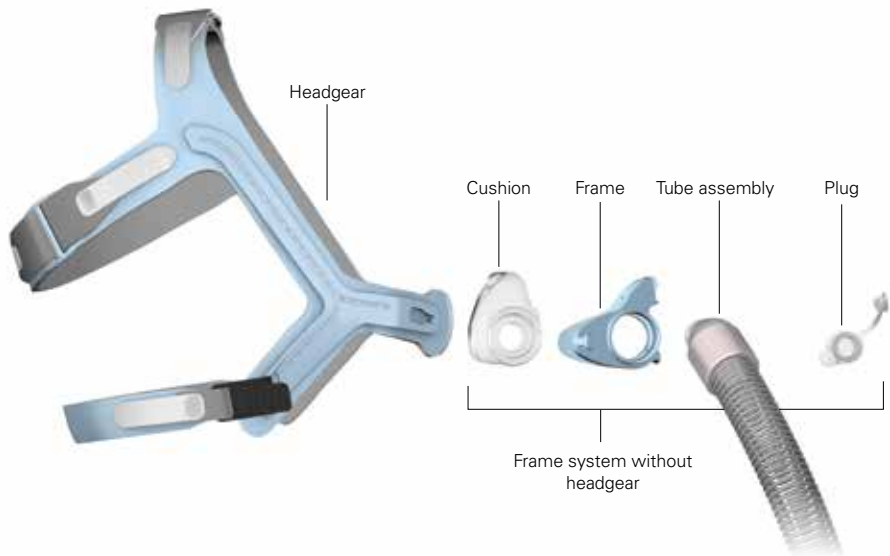

Pixi™

Paediatric mask

Component	Part Number
Cushion	61033
Headgear	61034
Frame system without headgear (includes frame, cushion, tube and plug)	61035

	Pixi™
EUR 1 & 2	61308
EUR 3	61032



**NON-VENTED
MASKS**



Quattro™ Air NV
Non-vented full face mask

Delivering NIV... the lighter way

A busy healthcare environment requires a mask that delivers high quality treatment with minimal fuss. Likewise, a patient on long-term, non-invasive ventilation (NIV) requires a mask that's comfortable and easy-to-use. ResMed's Quattro™ Air NV does it all in a lightweight mask designed to meet a range of needs both in the hospital and the home.

Comfort

The Soft-Edge™ headgear is made of premium rolled-edge fabric that's gentle on the skin. Its lightweight design may also help minimise skin breakdown to enhance patient comfort over long-term use.


Efficiency

Quattro Air NV has been designed to make initiating NIV easy and effective. Its Spring Air™ cushion is designed to provide a secure seal throughout the mask.

Simplicity

With only five parts, the mask can be assembled in minimal time. And thanks to its quick-release clip with cord, it can be removed quickly and easily in an emergency.



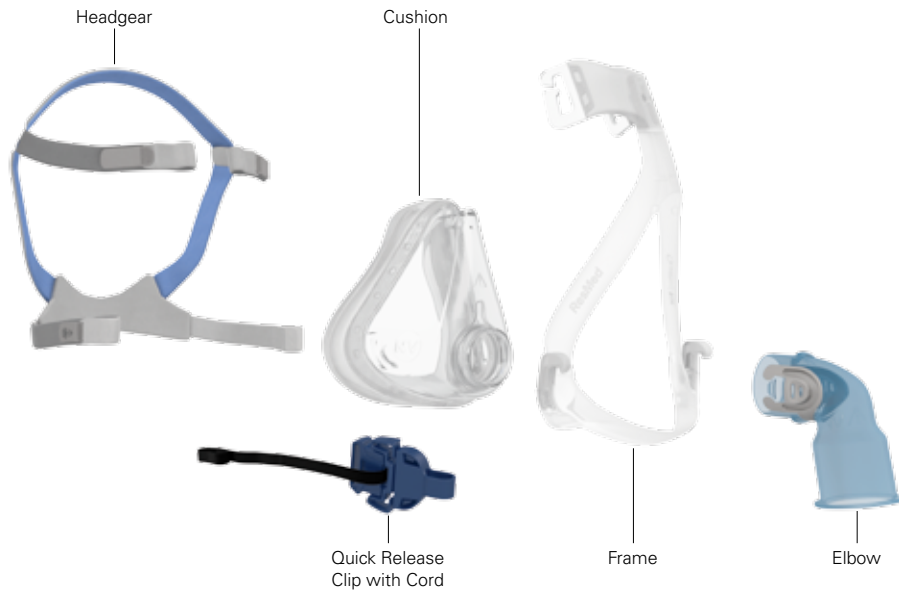

Quattro™ Air NV

Non-vented full face mask

Component		Part Number
Headgear	(S)	62821
	(Std)	62820
Cushion	(XS)	62823
	(S)	62824
	(M)	62825
	(L)	62826
Frame	(XS/S)	62760
	(M/L)	62761
Elbow		62827
Quick Release Clip with Cord		62848

Quattro™ Air Non-Vented				
Size ^{1, 2}	Small	Medium	Large	Wide
EUROPE	62782	62783	62784	62785

¹The XS mask system comes with the small sized headgear.

²The S, M, & L mask systems come with the standard size headgear.



Quattro™ FX NV

Non-vented full face mask

Therapy has never been easier

Light and unobtrusive, Quattro™ FX Non-Vented (NV) Full Face Mask provides reliability and comfort for noninvasive ventilation in the hospital or the home.

Clear line of sight and feeling of openness

created by a full face mask that starts mid-way on the nasal bridge.

Contoured cushion membrane around sensitive nasal bridge area

rolls and inflates to gently meet the nose rather than stretch over it.

Clear (non-tinted) frame enables the caregiver to more easily monitor

the patient's face for changes during therapy.

Flexible, durable Spring frame strong enough to stabilise the mask on the face

soft enough to accommodate user movement.

360° elbow rotation

positions the tubing for ease of use.

Blue-tinted elbow for added safety

specifically designed so that it cannot be interchanged with the clear elbow of ResMed's vented masks.


Dual-wall, Spring Air™ cushion

provides soft, even pressure distribution.

Set-and-forget headgear clips easy to grip for attaching and detaching

retain desired headgear settings.



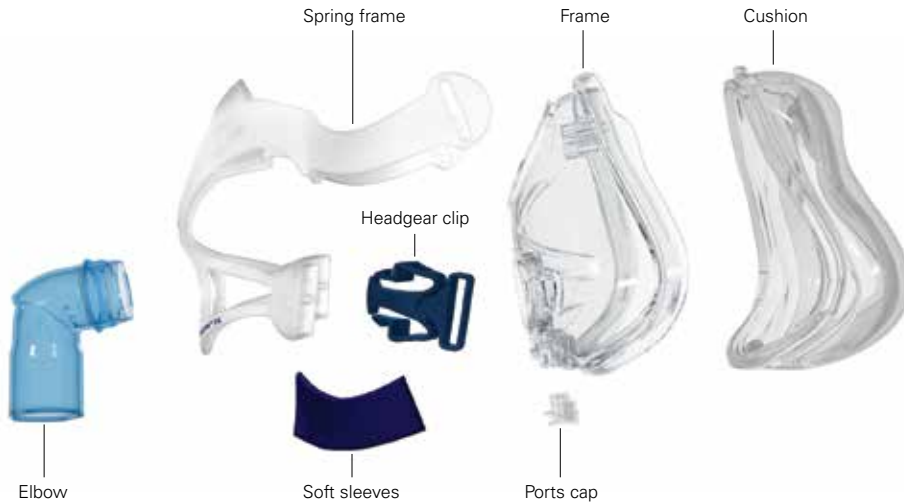

	Quattro™ FX Non-Vented		
Size	Small	Medium	Large
EUR 1	61742	61743	61743
EUR 2	61749	61750	61751
EUR 3	61752	61753	61754

All mask systems come with the Medium (M) headgear.

Quattro™ FX NV

Non-vented full face mask

Component		Part Number
Frame	(S)	61745
	(M)	61746
	(L)	61747
Soft sleeves		61758
Cushion	(S)	61721
	(L)	61722
	(M)	61723
Spring frame	(S)	61759
	(M)	61760
	(L)	61761
Clips		61753
Ports cap	(x2)	16570
	(x10)	16571
Headgear	(Std)	61733
	(S)	61734
	(L)	61738
Elbow		61762



Ultra Mirage™ NV

Non-vented full face mask

Stable choice for non-invasive ventilation

The Ultra Mirage Non-Vented mask range provides superior seal, stability and comfort for long-term non-invasive ventilation in the home as well as hospital environment.

Mirage dual-wall cushion

For proven comfort and seal.

Easy-access pressure and oxygen ports

Provide convenient monitoring and delivery of oxygen.

360° rotating elbow

Provides flexibility in the tube position for freedom of movement.

Adjustable forehead support and pads

Provide maximum stability and comfort.

Quick release headgear clips with emergency release

Allows headgear to be removed without refitting.

Easy and quick-to-fit headgear

Offers the ultimate in convenience and ease of set up.

Non-vented blue elbow

Allows easy identification as a non-vented mask.





Ultra Mirage™ NV

Non-vented full face mask

Component		Part Number
Forehead pad/s	(x2)	16561
	(x10)	16574
Forehead support		16562
Frame	(S)	60675
	(M)	60676
	(L)	60677
Elbow		60678
Headgear		60917
Tube connector/s	(x1)	60913
Emergency release cord assembly		60618
Straight connector		60699

Ultra Mirage™ NV full face mask						
Size	Small Standard	Small Shallow	Medium Standard	Medium Shallow	Large Standard	Large Shallow
EUR 1	60643	60642	60645	60644	60647	60646
EUR 2	60648	60649	60650	60551	60652	60653

European user information available in the following languages:

EUR 1: English, German, French, Spanish, Italian, Portuguese, and Dutch.

EUR 2: English, Swedish, Norwegian, Danish, Finnish.



Ultra Mirage™ NV

Non-vented nasal mask

Stable choice for non-invasive ventilation

The Ultra Mirage Non-Vented mask range provides superior seal, stability and comfort for long-term non-invasive ventilation in the home as well as hospital environment.

Mirage dual-wall cushion

For proven comfort and seal.

Easy-access pressure and oxygen ports

Provide convenient monitoring and delivery of oxygen.

360° rotating elbow

Provides flexibility in the tube position for freedom of movement.

Adjustable forehead support and pads

Provide maximum stability and comfort.

Quick release headgear clips with emergency release

Allows headgear to be removed without refitting.

Easy and quick-to-fit headgear

Offers the ultimate in convenience and ease of set up.

Non-vented blue elbow

Allows easy identification as a non-vented mask.




Ultra Mirage™ NV

Non-vented nasal mask

Component		Part Number
Forehead pad/s	(x2)	16561
	(x10)	16574
Forehead support		16562
Frame		60915
Elbow assembly		60916
Headgear		60917
Tube connector/s	(x1)	60913



Ultra Mirage™ Non-Vented nasal mask			
Size	Standard	Large	Shallow
EUROPE	60401	60402	60403

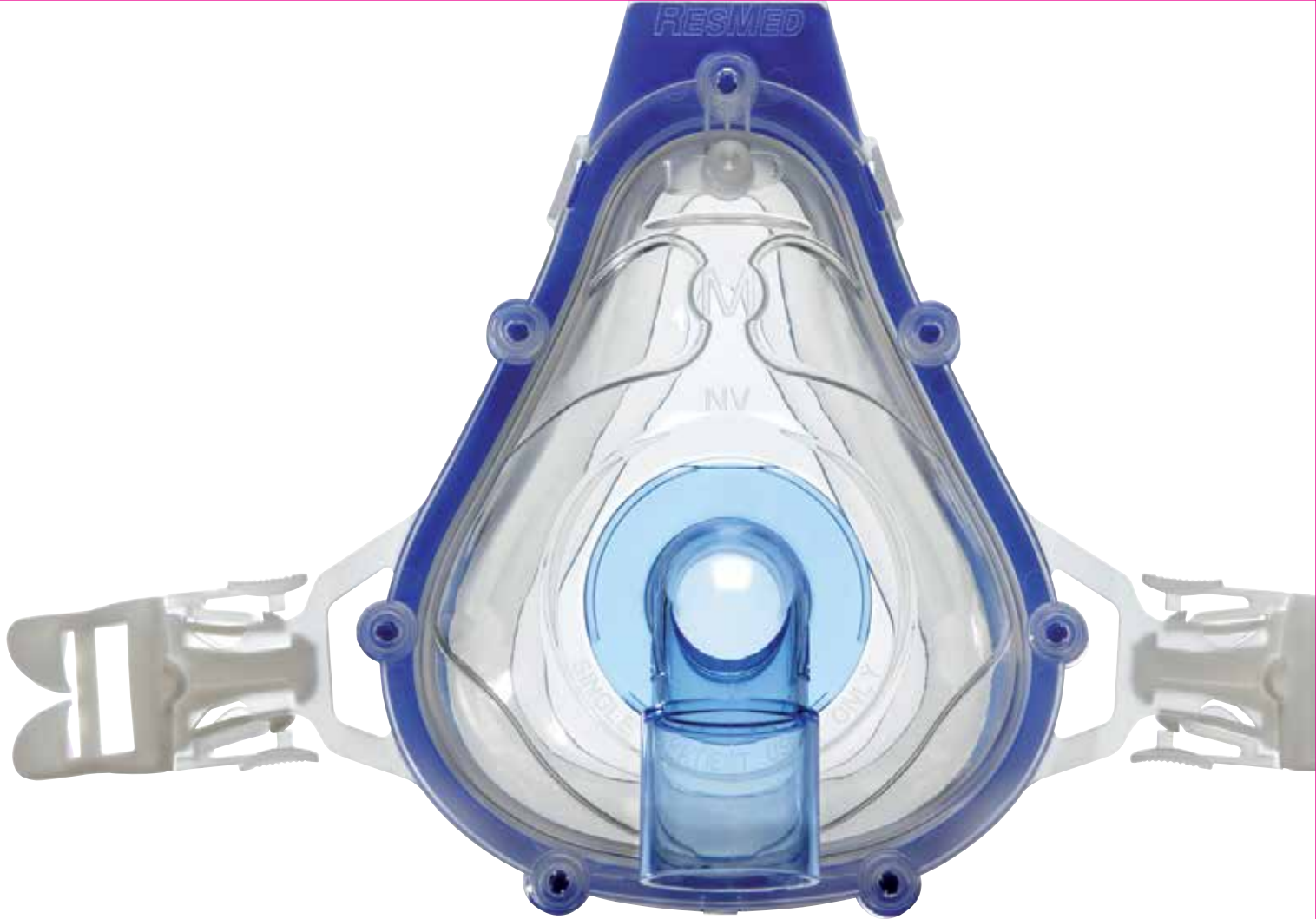



ResMed

Hospital

Full face mask

Quick-fitting, High Performance

An economic and disposable mask providing a 7-day solution for non-invasive ventilation in the acute setting where cross-contamination between patients presents a risk.

Mirage dual-wall cushion

For proven comfort and seal.

Pressure port

Single port to measure pressure.

360° rotating elbow

Provides flexibility in the tube position for freedom of movement.

Quick release headgear clips

Allow headgear to be removed without refitting.

Easy and quick-to-fit headgear

Offers the ultimate in convenience and ease of set up.



Hospital Full Face Mask			
Size	Small	Medium	Large
EUROPE	60700	60701	60702

Component	Part Number
Headgear	(x10) 60710



ResMed

Hospital

Non-vented full face mask

Quick-fitting, High Performance

An economic and disposable mask providing a 7-day solution for non-invasive ventilation in the acute setting where cross-contamination between patients presents a risk.

Mirage dual-wall cushion

For proven comfort and seal.

Pressure port

Single port to measure pressure.

360° rotating elbow

Provides flexibility in the tube position for freedom of movement.

Quick release headgear clips

Allow headgear to be removed without refitting.

Easy and quick-to-fit headgear

Offers the ultimate in convenience and ease of set up.



	Hospital Non-Vented Full Face Mask		
Size	Small	Medium	Large
EUR 1	60720	60721	60722
EUR 2	60726	60727	60728
EUR 3	60729	60730	60731

Component

Part Number

Headgear (x10) 60711

Quick-fitting, High Performance

An economic and disposable mask providing a 7-day solution for non-invasive ventilation in the acute setting where cross-contamination between patients presents a risk.

Mirage dual-wall cushion

For proven comfort and seal.

Pressure port

Single port to measure pressure.

360° rotating elbow

Provides flexibility in the tube position for freedom of movement.

Quick release headgear clips

Allow headgear to be removed without refitting.

Easy and quick-to-fit headgear

Offers the ultimate in convenience and ease of set up.



	Hospital Nasal Mask	
Size	Medium	Large
EUROPE	61104	61105

Component	Part Number
Headgear	(x10) 61118



ResMed

Bella™ Loops
Mask accessories

Interchangeable with Swift FX

Swift™ FX Bella Series headgear is fully interchangeable with Swift™ FX (both use the same pillows and tubing) for an increased level of patient compliance.



Bella™ Loops	
Bella Headgear assembly (pink)	61581
Bella Headgear assembly (gray)	61582
Headgear and Swift FX headgear Combo (pink)	61580
Headgear and Swift FX headgear Combo (gray)	61583



ResMed

Gecko™
Nasal pad

The Gecko nasal pad is a soft, comfortable strip placed across the nasal bridge to help:

- Reduce skin irritation and facial sores
- Minimize mask leak
- Improve overall mask comfort



	Gecko™	
Size	Small	Large
1 pack	61912	61913
10 pack	61916	91917



CPAP Wipes

Mask accessories

ResMed CPAP Mask Wipes are a ready-to use, fast and easy cleaning solution adapted for any CPAP mask or device.

Coming in a handy 62 wipes container for your patient daily home usage. Or in a 14 single wipes sachets perfect for travels. To ensure good hygiene at home and away, ResMed CPAP Mask Wipes are:

- 100% cotton wipes
- alcohol & latex free
- with a fresh unscented cleaning agent
- and aloe vera.

Only for home use



CPAP Wipes	
14 wipes	61919
62 wipes	61918

Care and cleaning

The most effective therapy is delivered through a clean, well-cared for mask. An unclean cushion, skin irritation and over stretched headgear may indicate a patient's therapy will be less effective and often cause them to wake during the night. We recommend cleaning the mask regularly to ensure optimum therapy.

Daily

- Hand wash the mask system components (cushion, elbow, frame, etc.) in warm, soapy water.
- Do not use soaps with added scents. We recommend plain liquid soap or baby shampoo.
- Rinse well and allow to air dry out of direct sunlight.

Weekly

- Hand wash the headgear in warm, soapy water.
- Do not use soaps with added scents. We recommend plain liquid soap or baby shampoo.
- Rinse well and allow to air dry out of direct sunlight.

Cautions

- Do not use solutions containing vinegar, bleach, chlorine, alcohol, aromatics, moisturizers, antibacterial agents or scented oils to clean any part of the system or air tubing. These solutions may cause damage and reduce the life of the product.
- Do not expose any part of the system or tubing to direct sunlight as it may deteriorate the product.
- Do not wash the mask in the dishwasher.
- Do not iron the headgear.

If any visible deterioration of the mask or a component is apparent (cracking, tears, etc.) the mask or a component should be discarded and replaced.

Technical information



Disinfection and sterilisation guide

Clinical use only

This guide is intended for multi-patient use of a ResMed full face mask, nasal mask or nasal pillow system (mask) in a sleep lab, clinic or hospital. If you use the mask as a single user in the home, refer to the User Guide for cleaning instructions. This guide describes ResMed's recommended and validated procedures for cleaning, disinfection and sterilisation of the mask in accordance with ISO 19846. Note: Only masks listed in the table below have been validated for reprocessing between patients.

Mask*	Replace these parts with new parts between patients	High level thermal disinfection EN ISO 15853 220°C for 30 min EU C1 class, BS C1 class European regional requirements 50°C-10 min	High level chemical disinfection Anolyte™ COD™/NA CIDE™ Plus 0.1% for 30 minutes absolute 0.5% for 20 minutes 2.4% for 20 minutes	Sterilisation		Validated number of cycles [†]
				1055	10X	
Full face masks						
Mirage Liberty	Sealant, inlet tube	✓	✓	✓	✓	20
Mirage Quattro [†]	None	✓	✓	✓	✓	20
Quattro FX/Quattro FX for Nasal [†]	None	✓	✓	✓	✓	20
Quattro FX NP [†]	None	✓	✓	✓	✓	20
Ultra Mirage	Valve membrane	✓	✓	✓	✓	15
Ultra Mirage Non-vented	None	✓	✓	✓	✓	15
Mirage Series 2	Valve membrane	✓	✓	✓	✓	15
Mirage Non-vented Series 2	None	✓	✓	✓	✓	15
Quattro Air / Quattro Air for Nasal [†]	None	✓	✓	✓	✓	20
Nasal masks						
Mirage	Sealant, inlet tubing	✓	✓	✓	✓	15
Mirage Active	Sealant, inlet tube	✓	✓	✓	✓	15
Mirage Active CP	None	✓	✓	✓	✓	20
Mirage FX / Mirage FX for Nasal [†]	None	✓	✓	✓	✓	20
Mirage SoftGel [†]	None	✓	✓	✓	✓	20
Mirage Kolibri	Sealant, inlet tube	✓	✓	✓	✓	15
Mirage Liberty	None	✓	✓	✓	✓	20
Mirage Vito	Sealant, inlet tube	✓	✓	✓	✓	15

Page 1



Mask/Device Compatibility List

This document specifies the masks compatible with ResMed's positive airway pressure devices and the mask setting, if one is required. For full details regarding the correct use of these devices, please refer to the specific User Guides.

Note: Not all products are available in all regions.

Full Face Masks

Device	Mirage™	Mirage Series 2	Ultra Mirage™	Mirage Quattro™	Quattro™ FX / Quattro™ FX for Nasal	Quattro™ Air / Quattro™ Air for Nasal	Mirage Liberty™	Hospital Full Face
SD Aspire™ / SD Elite™	NV	NV	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face
SD Escape™	NV	NV	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face
Aspire CP 2	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face
Aspire CP 200	NV	NV	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face
Aspire CP 200 Resistant / Aspire USA™ / SD Resistant	NV	NV	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face
CPlex™ Single	No setting required. Consult your clinician before changing to a different mask type.							
ST 100™ Single	MRR Full	MRR Full	MRR Full	MRR Full	MRR Full	MRR Full	MRR Full	MRR Full
ST Elite™ / ST Aspire™ Single	STANDARD	ST 100	MIRAGE	MIRAGE	MIRAGE	MIRAGE	MRR Full	MRR Full
ST Aspire 20	MRR Full	MRR Full	MRR Full	MRR Full	MRR Full	MRR Full	MRR Full	MRR Full
SD Aspire™ Single	MRR Full	MRR Full	MRR Full	MRR Full	MRR Full	MRR Full	MRR Full	MRR Full
SD Aspire™ Single™	MRR Full	MRR Full	MRR Full	MRR Full	MRR Full	MRR Full	MRR Full	MRR Full
SD Escape™ / SD Escape™ 2	MRR Full	MRR Full	MRR Full	MRR Full	MRR Full	MRR Full	MRR Full	MRR Full
SD Elite™ / SD Elite™ 2	MRR Full	MRR Full	MRR Full	MRR Full	MRR Full	MRR Full	MRR Full	MRR Full
SD Aspire™ 2	MRR Full	MRR Full	MRR Full	MRR Full	MRR Full	MRR Full	MRR Full	MRR Full
SD Comfort™	MRR Full	MRR Full	MRR Full	MRR Full	MRR Full	MRR Full	MRR Full	MRR Full
VPAP™ Bi™	MRR Full	MRR Full	MRR Full	MRR Full	MRR Full	MRR Full	MRR Full	MRR Full
VPAP™ Bi™ A	MRR Full	MRR Full	MRR Full	MRR Full	MRR Full	MRR Full	MRR Full	MRR Full
VPAP™ Bi™ CP	MRR Full	MRR Full	MRR Full	MRR Full	MRR Full	MRR Full	MRR Full	MRR Full
VPAP™ Aspire™ ST	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face
VPAP™ Aspire™ ST	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face
VPAP™ Adapt™ ST	NV	NV	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face

Page 1

For more detailed technical information on our masks please refer to the Service and Support section on our website:

- Mask Disinfection and Sterilisation
- Mask/Device Compatibility
- Frequently Asked Questions
- User & Clinical Guides

www.resmed.com/uk/service_and_support

Technical innovation

There's innovative technology behind every mask



Dual-Wall Cushion

- Thick inner wall provides rigidity and stability.
- Thin outer membrane offers comfortable seal.
- Airflow creates an air layer between the wall and the membrane, providing a comfortable and effective seal with minimal headgear tension.
- Dual-wall cushion provides even pressure distribution with extra softness at the nasal bridge, reducing common discomfort in this sensitive area.



QuickFit™ elastic headgear

- With unique split-strap design provides stability and adjustability with minimal facial contact.



Spring Air™ Technology

- Cushion geometry creates a “spring” effect and optimises seal due to the built-in elasticity of the cushion.
- Adapts to different facial contours and accommodates movement during sleep, including jaw drop.



QuietAir™ woven-mesh venting

- Quietly diffuses exhaled air and is barely noticeable to patients and their bed partners.
- The AirFit P10 is ResMed's quietest mask at just 21.4 dBA.

Technical innovation



Anti-asphyxia Valve (AAV)

- Ensures safety for the patient.
- Prevents CO₂ re-breathing when airflow is impeded.
- Integrated in all ResMed vented, full face masks.



Flex-Wing Forehead Support

- Is comfortable and feels light on the face.
- Allows a dynamic range of fit and adjustment.
- Streamlined frame design eliminates the need for extra parts.



SoftEdge™ Headgear

- Materials are flexible and breathable, improving comfort and support.
- Kind to the face-made from soft materials with a rounded cross-section, reducing marks and discomfort.
- New, form-fitting headgear springs to life, ready to fit and wear.



Quick release elbow

- Innovative squeeze-tabs make attaching and detaching quicker and easier.
- Tube can be easily attached to the mask with one hand.
- A smoothly rotating elbow with a swivel can freely rotate 360°.
- Its all-around flexibility accommodates just about every sleeping position, helping to maintain seal and reduce tube drag.



Europe 1: English (UK), French, German and Italian (Switzerland)

Europe 2: English, Danish, Finnish, Estonian, Norwegian and Swedish

Europe 3: English, Czech, Dutch (Belgium), French (Belgium), German (Belgium), Greek, Hungarian, Italian, Polish, Portuguese, Russian, Slovak, Spanish and Turkish



ResMed Ltd
1 Elizabeth Macarthur Drive,
Bella Vista NSW 2153 Australia

DISTRIBUTED BY

EC REP ResMed (UK) Ltd 96 Jubilee Avenue
Milton Park Abingdon Oxfordshire OX14 4RW UK

Quattro, Mirage, Swift, QuickFit, QuietAir, MicroFit, EasyClick, AirFit, SoftEdge and Spring Air are trademarks of ResMed Ltd. SoftEdge and Spring Air Technology are registered in the U.S. Patent and Trademark Office. Product availability may vary across regions. See www.resmed.com for other ResMed locations worldwide. © 2014 ResMed Ltd. Specifications may change without notice. 1017258 - 2014-02

